EN

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CHEMSTAR® BY GRANBERG

ART. 114.2000

The EN ISO EN 374-1:2016 test measures the BTT penetration time that indicates the time a chemical substance takes to penetrate the glove material. The penetration time is defined in standard EN ISO 374-1:2016 as the rate of permeation of a chemical substance through the glove sample that is equivalent to 1 micro-gram per square centimetre and per minute. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only. This information does not reflect the usual working condition in the workplace and the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 40 cm² - where the cuff is tested also) and relates only to the chemical tested. It can be assumed that the chemical resistance is higher. Working protective gloves may provide better resistance to the dangerous chemicals due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For certain chemicals, the resistance factor to consider is the factor to compare in selecting of chemical resistant gloves. It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

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CHEMSTAR® BY GRANBERG

ART. 114.2000

The EN ISO EN 374-1:2016 test measures the BTT penetration time that indicates the time a chemical substance takes to penetrate the glove material. The penetration time is defined in standard EN ISO 374-1:2016 as the rate of permeation of a chemical substance through the glove sample that is equivalent to 1 micro-gram per square centimetre and per minute. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 40 cm² - where the cuff is tested also) and relates only to the chemical tested. It can be assumed that the chemical resistance is similar. Work protective gloves may provide some resistance to the dangerous chemicals due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For certain chemicals, the resistance factor is a factor to consider in selecting of chemical resistant gloves. It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

Level 0 - the glove is below minimum level of individual risk given.

Level X - the glove has not undergone the test or the test method is not suitable for the design or glove material.

The results are taken from the palm area of the gloves.

The protection level increases with performance class.

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4/B = maximum requirement

X = not tested

The results are taken from the palm area of the gloves.

The protection level increases with performance class.

EN ISO 374-5:2016 EN 420:2003+A1:2009 EN ISO 374-5:2016

AQL 1.5

This product is classified as Category III Personal Protective Equipment (PPE) according to PPE Regulation (EU) 2016/425 and has been shown to comply with this Regulation through the Harmonised European Standards EN 388:2016, EN 420:2003+A1:2009, EN ISO 374-5:2016 and EN ISO 374-2:2016.

PPE CAT. III: Complex design PPE that protects against irreversible risks or injuries with mortal danger or that could cause very serious injuries. Levels only refer to the palm of the hand.

Notified Body responsible for certification and Module B compliance:

Name: S.A.T. Technology Europe Ltd.

Address: Broomhill Business Park, Clonee, Co. Meath, D15 YN2P, Ireland.

Notified Body: 2777

Notified Body responsible for ongoing conformity (Module D):

Name: S.A.T. Technology Europe Ltd.

Address: P.O. Box 30 (Särkkämienteenva 3), 00211 Helsinki, Finland.

Notified Body: 0598

Declaration of conformity can be obtained from www.granberg.no/search

Glove size M L XL XXL

EN 420 size 8 9 10 11

Wear the products of suitable sizes to provide optimal level of protection and maximum grip. The user must only choose the glove fitting the size of her/his hand.

Barcode

</div

EN

Neoprene unsupported glove, flocked, 30 cm. Protects against several chemicals.

Storage/Transport: The gloves are first packed by the dozen in plastic bags which are subsequently packed in cardboard boxes for transport and storage. Granberg recommends storing unused gloves in original packaging. Prevent direct sunlight.

Maintenance/Cleaning: Both new and used gloves should be inspected before use, and before putting them on to make sure that there is no damage to them. If in doubt, discard the gloves and select a new pair. Leaving the gloves in a contaminated condition may cause a deterioration of quality. Cleaning or disinfecting the gloves may also negatively affect quality. The performance characteristics of gloves that have been worn or cleaned/disinfected/laundered may differ from the performance levels declared. Gloves can only be cleaned with damp cloth, but it will not stop permeation processes.

Obsolescence: When stored as recommended, unused gloves will not suffer any change in their mechanical properties for up to five years from the date of manufacture. Service life depends on application and maintenance and cannot be specified. It is the responsibility of the user to ascertain the suitability of the gloves for the user's task or tasks.

Please note: The gloves should be inspected for any damage before use. The gloves shall not be worn when there is a risk of entanglement with moving parts of machines. Gloves meeting the requirement for resistance to puncture may not be suitable for protection against sharply pointed objects such as hypodermic needles. Gloves containing latex may cause allergic reactions in cases of hypersensitivity to latex. Seek medical advice if applicable. None of the raw materials used in glove, or process of manufacturing it, are known to be harmful to the user. These gloves are intended to protect hands in working environment in accordance with EN 388:2016, EN ISO 374-1:2016/Type A, EN ISO 374-5:2016 and EN 420:2003+A1:2009. It is the responsibility of the user to evaluate and determine risks based on the intended application. The gloves should only be used for applications declared suitable by the manufacturer. Risk should be evaluated keeping in mind the protection levels and the harmonized standards on which gloves are tested. The results of the tests should help in glove selection, however it must be understood that actual conditions of use cannot be simulated and it is the responsibility of the user, not the manufacturer, to determine glove suitability to the intended use. Further information may be obtained from manufacturer.

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CHEMSTAR® BY GRANBERG

ART. 114.2000

The EN ISO EN 374-1:2016 test measures the BTT penetration time that indicates the time a chemical substance takes to penetrate the glove material. The penetration time is defined in standard EN ISO 374-1:2016 as the rate of permeation of a chemical substance through the glove sample that is equivalent to 1 micro-gram per square centimetre and per minute. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only. This information does not reflect the usual working conditions in the workplace and the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 40 cm² - where the cuff is tested also) and relates only to the chemical tested. It can be assumed that the chemical resistance is higher. Working protective gloves may provide better resistance to the dangerous chemicals due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For certain chemicals, the resistance factor to consider is the factor to compare in selecting of chemical resistant gloves. It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

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CHEMSTAR® BY GRANBERG

ART. 114.2000

The EN ISO EN 374-1:2016 test measures the BTT penetration time that indicates the time a chemical substance takes to penetrate the glove material. The penetration time is defined in standard EN ISO 374-1:2016 as the rate of permeation of a chemical substance through the glove sample that is equivalent to 1 micro-gram per square centimetre and per minute. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 40 cm² - where the cuff is tested also) and relates only to the chemical tested. It can be assumed that the chemical resistance is similar for other chemicals due to changes in the workplace and the different between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 40 cm² - where the cuff is tested also) and relates only to the chemical tested. For certain chemicals, the resistance factor to consider is the difference between the resistance of the glove and the resistance of the pure chemical. It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

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AQL 1.5

AKLMOPT

This product is classified as Category III Personal Protective Equipment (PPE) according to PPE Regulation (EU) 2016/425 and has been shown to comply with this Regulation through the Harmonised European Standards EN 388:2016, EN 420:2003+A1:2009, EN ISO 374-5:2016 and EN ISO 374-2016.

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Name: SGS Technology Europe Ltd.
Address: Bracknell Business Park, Clonée, Co. Meath, D15 YN2P, Ireland.

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