## KARTA KATALOGOWA

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## DATA SHEET

# Filtering half mask XA 300V FFP3 NR D

The XA 300V FFP3 NR D filtering half-mask is designed to protect the respiratory system against aerosols of solid particles (dust) and liquid particles (mists) as long as the concentration of the harmful substance does not exceed 30 times the Maximum Permissible Concentration up to - 30 x TLV.

### EXAMPLES OF APPLICATION:

Medium toxic solids, asbestos, copper, barium, titanium, vanadium, chromium, manganese, hardwood, coal dust with free silica content higher than 10%, mining industry, chemical industry, metallurgic industry, welding, soldering, respirable dusts.

### DESCRIPTION:

The filtering half mask XA 300V FFP3 NR D is composed of the following elements:

- Bowl made of multi-layer filter material: polypropylene;
- Nose clip made of polypropylene tape reinforced with two steel wires;
- Exhalation valve
- Headbands
- The product is available in two colors: white or blue.

### REQUIREMENTS:

The XA 300V FFP3 NR D filtering half mask complies with:

- harmonised European standard EN 149:2001+A1:2009 "Respiratory protective devices - Filtering half masks to protect against particles. Requirements, testing, marking";
- in conformity with the relevant Union harmonisation legislation: Regulation (EU) 2016/425 of the European Parliament and of the Council of 9 March 2016 on personal protective equipment and repealing Council Directive 89/686/EEC (Text with EEA relevance)

**RECOMMENDATIONS AND RESTRICTIONS ON USE:** Read the instructions for use of the respirator.

1. Read the instructions for use of the half mask.
2. It is prohibited to use the half-mask in an atmosphere with an oxygen content below 17% by volume. and in confined spaces with a small volume.
3. The XA 300V FFP3 NR D filtering half mask is a disposable product (to be used for one shift) and is not intended for cleaning or disinfection.
4. Facial hair or other specific features of the face may reduce the effectiveness of the respirator.
5. Half masks must not be modified or altered in any way.
6. Do not use half masks if the type, concentration and properties of harmful substances are not known.
7. No maintenance required. After use, the mask must be disposed of in accordance with applicable laws and regulations.

### STORAGE:

The half-mask should be stored in the factory packaging, in dry rooms protected against precipitation, ensuring the temperature is maintained from -20°C to + 50°C and the relative humidity is approximately 80%, free from direct sunlight.

### DURABILITY:

**Shelf life 36 months when stored in the original factory packaging. The expiration date is located on the product.**

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### REQUISITOS:

La media mascarilla filtrante XA 300V FFP3 NR D cumple con:

- harmonised European standard EN 149:2001+A1: 2009 " Respiratory protective devices - Filtering half masks to protect against particles. Requirements, testing, marking";
- in conformity with the relevant Union harmonisation legislation: Regulation (EU) 2016/425 of the European Parliament and of the Council of 9 March 2016 on personal protective equipment and repealing Council Directive 89/686/EEC (Text with EEA relevance)

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